



DCD

PRODUCTS S.R.L.


Representante Exclusivo

Lartigau 1152 • (1875) Wilde
Avellaneda • Buenos Aires
Tel.: 4217-1223 • Fax 4217-1226
e-mail: dcdventas@speedy.com.ar

PAJUNK®

World Novelty:

*Fixolong –
the mobile fixation system
for catheters and filters.*

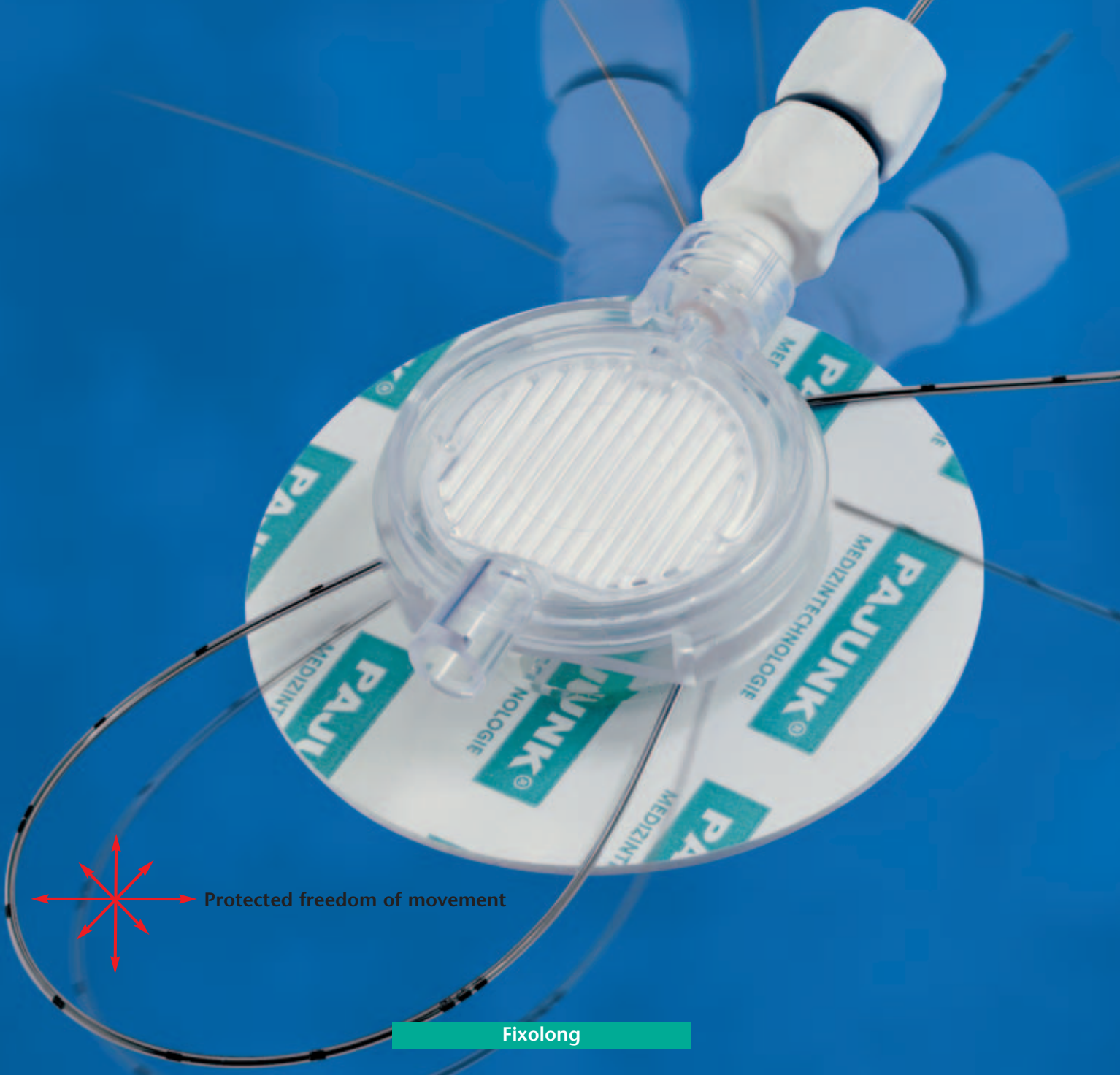

Protected freedom of movement

Fixolong

Protected freedom of movement with Fixolong



Conventional "method"

Till now, the way of fastening catheters and filters to the patient has been left up to the anesthetist. Problems arising especially during continuous application are to safeguard, that the catheter isn't inadvertently pulled out if the patient moves, or that the supply of the anesthetic is obstructed by an unfavorable position.

But now this will change. Because Pajunk has recognized the need for freedom of movement, while at the same time guaranteeing safe and secure fixation, and therefore Pajunk is offering the perfect solution for catheters of the size 19 and 20 G.

PAJUNK GmbH

Medical Technology

Karl-Hall-Strasse 1

D-78187 Geisingen/Germany

Telefon +49 (0) 77 04/92 91-0

Telefax +49 (0) 77 04/92 91-6 00

www.pajunk.com

Fixolong

Sticking plaster with catheter cross, filter adapter	Item no. 001151-40
Sticking plaster with catheter cross, filter adapter, filter	Item no. 001151-41

Step by step progress towards more freedom of movement




1. The Pajunk sticking plaster with the fixated catheter cross is fastened in the vicinity of the catheter exit.



2. The catheter is snapped onto the fastening clips. This guarantees maximum freedom of movement while simultaneously fixating the catheter.



3. The filter adapter is plugged onto the catheter cross in a fixed position.

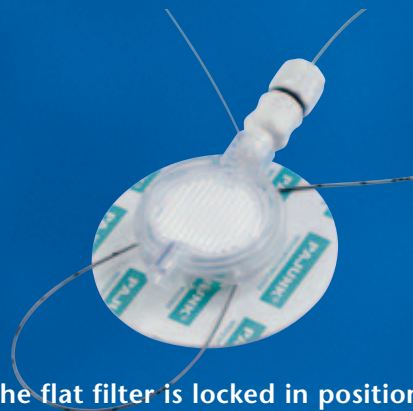

4. The flat filter is locked in position on the filter adapter.

The result: protected freedom of movement during all continuous applications.